A B C
User manual

FIRST AID KIT

small

NEXA

EN

L

L – Life-threatening situation

Check the scene and form an initial impression. Assess the situation and check for any potential hazards. If the situation is not safe and you cannot help without putting yourself in danger, call 112. Then wait for emergency personnel.

A

AIRWAY

Check the airway. If the injured individual is unresponsive, tilt the head back and lift the chin to open the airway.

• Unconscious – breathing normally; place in a stable lateral position. Continue to step B; bleeding.

• Unconscious – not breathing; call 112 and start cardiopulmonary resuscitation, CPR.

B

BLEEDING

• Stop external bleeding by applying pressure to the wound.

• Signs of internal bleeding; pain, pale and feels cold, rapid and shallow breathing.

C

CIRCULATION

If signs of shock, and in all major injuries, circulatory failure must be prevented.

Do this:

• Allow the person to lie down and try to keep the injured individual calm.

• Prevent hypothermia by placing blankets, sweaters, or jackets on and under the injured individual.

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Plaster

Intended use:

For use on minor cuts and abrasions. Protects minor wounds. Read instructions prior to use; retain for future reference.

8 Pcs
56x19mm

45125

Instruction for use:

Clean wound and surrounding area and dry skin before applying the adhesive bandage. Remove adhesive bandage from protective wrapper. Position pad over wound and press firmly on adhesive area to ensure adhesion. Change dressing every 24 hours or if it is wet. Keep dry and out of direct sunlight. Store below 25°C.

45126

2 Pcs 72x25mm

2 Pcs 72x19mm

Warning: Use immediately after opening protective wrapper. Wrapper is sealed with natural rubber latex. Do not use on major wounds. If wound bleeds excessively or skin becomes irritated or infected remove adhesive bandage/plaster. Not suitable for children under 2 years old. In rare cases, a reaction to the adhesive bandage/plaster may occur in small children with exceptionally sensitive skin. This may be due to heat, moisture and other factors when plaster is used especially in the face. In both cases consult a doctor immediately. If you are diabetic consult your doctor before use.

45171

Pair of scissors

Intended use:

For cutting plaster, bandage, gauze or other small first aid devices. The scissors have a rounded cutting edge in order to protect the user from self-inflicted injuries.

1 Pcs
9cm

Warning:

Keep out of reach of children

45280

Skin cleansing wipe

Intended use:

For cleaning the skin and area around minor wounds and abrasions.

Instruction for use: wipe carefully around the area to be cleaned.

4 Pcs
3x3cm

Warning:

External use only. Avoid any contact with the eyes, mucosa, genitals and injured skin. Contains 70% isopropyl alcohol. Do not use in case of allergies. Product to be used once only. Check the integrity of the individual sachet before use. In case of repeated discomfort, ask your doctor for advice. If swallowed, contact a doctor immediately and show the packaging. Use before the expiry date on the sachet. Store in a cool, dry place. After use, do not dispose of the compress in the environment.

45002

Dry Swab

Intended use:

For cleaning and drying the wound and the skin around the wound. Protecting a clean and dry wound.

Instruction for use:

Open the package and wipe gently in the area to be cleaned / dried. Or place the swab on the area that needs protection.

2 Pcs
15x7cm

Warning:

Product to be used once only. Check the integrity of the individual sachet before use. Store in a cool, dry place. Use before the expiry date on the individual sachet.

45111

Triangular bandage

Intended Use:

To immobilise and keep splints in place. Use this bandage as a sling to support an injured, sprained or broken arm.

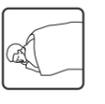
Instruction for use:

Unfold the bandage and attach around the neck as shown.

Warning:

Product to be used once only. Use before the expiry date on the individual item. Check the integrity of the individual item before use.

1pcs
136x96x96cm

45109

Emergency blanket

Intended use:

To protect against heat and cold. The emergency blanket can provide protection against heat, cold and humidity in emergency situations.

Instructions for use:

Emergency blanket to be used once only. Open the sachet and unfold the blanket as required.

Two sides with different functions:

The golden side absorbs heat and keeps you warm: providing a defence against potential hypothermia, maintaining your body temperature, and protecting you from the wind and humidity. The silver reflective outer side keeps you cool: keeps you chilled (cars, tents, etc.).

Warning:

Do not cover your head, danger of suffocating. Single use only. Do not use the emergency blanket during storms, near to incandescent substances or if using an electrical surgical device (e.g. DSA). Do not cut or create wholes. Store in a dry place.

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